



BACTRACK® SingleShot™


DISPOSABLE BREATH ALCOHOL TEST

CHANGE IN COLOR INDICATES
BLOOD ALCOHOL CONTENT

The BACtrack® SingleShot™ is a single-use disposable breath alcohol tester that is over 99% accurate and provides rapid results.


KEY FEATURES

- The SingleShot is less than three inches long and a quarter inch thick so it can literally be carried in any purse or pocket. Squeeze the center of the tube to activate the test, and blow through the SingleShot. Based on your Blood Alcohol Content (BAC), the crystals will either stay yellow, indicating a negative result, or change to blue/green, indicating a positive result.
- The BACtrack SingleShot is available at 0.02, 0.05, and 0.08 %BAC thresholds. The 0.02 test is perfect for zero-tolerance environments like schools, businesses, and for family use. The higher 0.05 and 0.08 thresholds can help indicate increased levels of intoxication help prevent alcohol-related accidents.
- The SingleShot does not require batteries, maintenance, recalibration or training. Each package contains two SingleShot tubes.

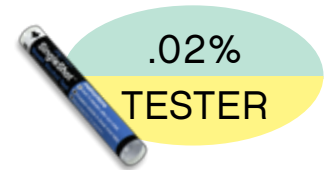


.02%

TESTER

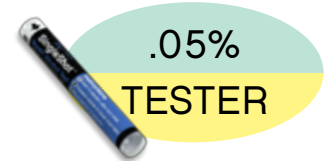

.05%

TESTER

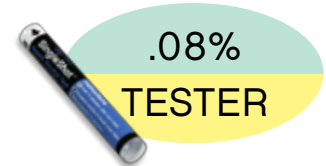

.08%

TESTER

Positive Result = Green/Blue

Negative Result = Yellow

FOR MORE INFORMATION

Toll Free: 1.877.334.6876
Online: www.bactrack.com
Email: info@bactrack.com

KHN Solutions LLC
300 Broadway, Suite 26
San Francisco, CA 94133 USA

BACTRACK® SingleShot™

TRACKS BLOOD ALCOHOL CONTENT IN SECONDS

Change in color indicates Blood Alcohol Content

- Simply blow through the tube and observe results
- Change in color indicates Alcohol Content above or below BAC threshold
- Available at 0.02, 0.05, and 0.08 %BAC thresholds
- FDA 510(k) cleared

PRODUCT SPECIFICATIONS


Dimensions.....	2.75 x .375 x .375 inches (6.9 x .9 x .9 cm)
Weight	0.2 oz (5.7 grams)
Sensor Technology.....	Alcohol Reactive Chemical Crystals
Blowing Time	12 seconds
Response Time.....	120 seconds
B.A.C. Range	0.02, 0.05, and 0.08 %BAC
Accuracy	99% Effective
Use.....	Single Use
Shelf Life.....	18 Months

KIT INCLUDES

- 2 disposable single use units
- One instruction insert

INTENDED USE

This device is intended to measure alcohol in human breath. Measurements obtained by this device are used in the diagnosis of alcohol intoxication.

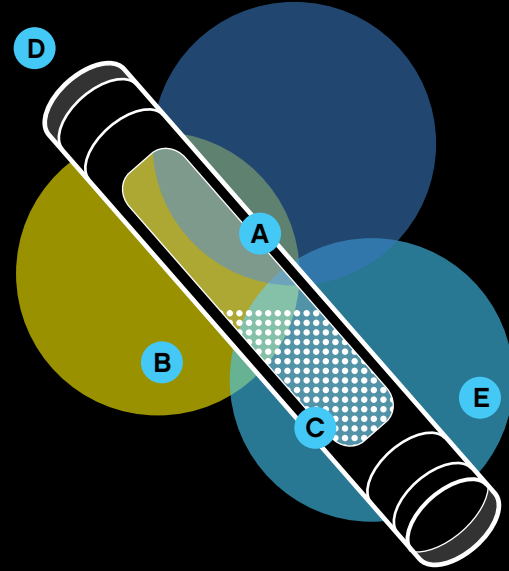

- A** A simple pinch activates the crystals – no warm up time necessary
- B** Miniature size makes it easy to carry or store anywhere
- C** Crystals clearly change from Yellow to Blue/Green when result is positive
- D** Safe and easy to blow into
- E** Cleared by the U.S. Food & Drug Administration and Department of Transportation – over 99% accurate

FOR MORE INFORMATION

Toll Free: 1.877.334.6876
Online: www.bactrack.com
Email: info@bactrack.com

KHN Solutions LLC
300 Broadway, Suite 26
San Francisco, CA 94133 USA