

Owner's Information



Model #11353

Congratulations on the purchase of your new

TouchAmerica Artesian Table

With proper care and maintenance, your new table will provide years of superior, trouble-free service. This document will help familiarize you with the equipment, outline your warranty, provide cleaning and maintenance instructions, and include contact information in the event you need technical support.

WARRANTY INFORMATION

Ten-year warranty on replacement parts for frame/structure and one year on labor. Foot tub and plumbing parts are warranted for one year on materials. Mechanical / electrical parts, foam and vinyl carry a two-year warranty on materials. Warranties apply to the original owner only. Labor warranty applies to customers in USA only. Customers are responsible for freight expenses associated with returning products to TouchAmerica.

CARE AND MAINTENANCE

Electrical: Plug the table into a high-quality Surge Protector to safeguard the circuits and controls. Do not attempt to operate all three motors at one time.

Leveling: If table is not level, adjust one or more of the leveling feet up or down, as needed.

Cleaning: Vinyl – Only non-abrasive, water-based cleaning agents and foam may be used for cleaning these fabrics. As with any vinyl, it is extremely important to pretest your cleaning method on a hidden area of the fabric to determine if the color bleeds or if the fabric shrinks.

Never use acetone or harsh chemicals to clean any surface of the table! When in doubt, seek the help of a professional cleaning service. To protect and extend the life of the vinyl, we recommend covering it with a sheet or paper roll.

IMPORTANT!

Draining Foot Tub: Drain foot tub as soon as possible after use. Allowing water to stand in tub for more than two hours while covered by padded foot section can cause mold or other damage to plywood backing, and will **void the warranty** on this structure.

Maintenance: In order to keep your table in top condition and to maintain your warranty, an inspection/tune-up must be performed twice yearly by someone with general mechanical ability. Connections may naturally loosen over time due to humidity changes or various pressures the table will be exposed to. Therefore, a routine preventive maintenance schedule should be followed. See Preventive Maintenance Instructions included in this packet.

Self-repair (beyond preventive maintenance) or modification of your table without prior factory approval may void your warranty! Always check with our technical support department first.

CAUTIONS

When moving the table, **do not** lift it by grasping the upholstered pads at the head and foot sections of the table. Instead, reach below and lift the table by its wooden under-frame. Failing to observe this method will potentially tear free the two motors which lift the head and foot sections, damaging your table. Also take care not to set the table on one of the electrical cords connecting the control box to the wall and to the pedestal motor, which could sever them. Should you accidentally disconnect one of these cords, you will find that significant force will be required to reconnect it and restore use of the table. This is designed as a safety feature.

Lift the table slightly when moving it. Dragging the table across the floor can damage the leveling feet. Your table weighs 250 pounds. Always have someone help you when moving it.

The lifting weight capacity for this table is 375 lbs. and the static capacity is

800 lbs. Include the considerable weight of water in the foot tub when calculating the load on the table's motor.

It is unsafe for you or your clients to sit on the outermost edge of this table at the head and foot sections. Please avoid sitting or placing heavy objects on those sections when they are elevated. Do not sit on the optional face cradle when it is inserted into the table, or push off against it when rising from the table: it is not designed to hold your full body weight.

OBTAINING TECHNICAL SUPPORT

Should you have a technical question or concern regarding your table, please email info@touchamerica.com with a complete description and photos. For warranty purposes, the shipping date is considered to be the date of purchase. Be sure to retain your packing list or invoice, which indicates shipping date, as well as model number.

PARTS LIST

Dewert Parts

Head Top Motor	98-77015
Thigh Top Motor	98-77006
Base Motor	98-07016
Control Box	98-79000
Hand Control	98-73005

Linak Parts

Head Top Motor	98-07015
Thigh Top Motor	98-07006
Base Motor	98-07016
Control Box	98-09000
Hand Control	98-13005

Tub	998-024
-----	---------



TouchAmerica

evolving the spa experience

www.touchamerica.com

(800) 678-6824 · (919) 732-6968