



Home of the Best Urology Products for Men and Women

We Offer the most advanced FDA and CE certified Vacuum Erection Systems in the world.

What is a Vacuum Erection Device (Penis Pump)?

Vacuum Erection Systems (VED) are designed to help millions of men suffering from Erectile Dysfunction with greater than 90% satisfaction in various studies.

More than 40 million American men suffer from ED of various causes. Medical grade VED systems are an excellent drug-free and non-invasive option to obtain satisfactory erection. At Urology Health Store, we offer more than 8 Medical VED options by the two most respected medical device manufacturers in the world.

Vacuum Erection Device is a medical pump, which creates a vacuum in the cylinder around the penis. This draws blood in the shaft of the penis and causes it to swell and become erect. One of the constriction rings is then slid down over the shaft of the penis. This prevents blood from flowing back out of the penis when the cylinder is removed and helps maintain erection. **Please look inside this brochure for step by step details** for proper application of this effective treatment.

DAILY LIVE VIDEO VED INSTRUCTIONAL CLASSES

We offer **FREE** 15-30 minute convenient individual or group **LIVE video VED instructional classes** via SKYPE, Go-To-Meeting or FaceTime.

Available by appointment from 1 - 3pm U.S. Eastern time, Monday - Friday.
Call 301.378.8433 for appointment.
No purchase necessary.

Ask your Urology Care provider for
10% DISCOUNT PROMOTIONAL CODE
at UrologyHealthStore.com

WWW.UROLOGYHEALTHSTORE.COM

Reflexonic, LLC
915 Toll House Avenue • Suite 209 • Frederick, Maryland 21701
Tel: 301.378.8433 • Fax: 800.886.1470 • customerservice@urologyhealthstore.com



Home of the Best Urology Products for Men and Women

VACUUM ERECTION SYSTEMS (VED)

For Management of Erectile Dysfunction (ED)



Made in USA



- ▶ High-Performance FDA & CE Certified Products
- ▶ Discounted Pricing
- ▶ Free U.S. Shipping on orders of \$50 or more

TO ORDER: UROLOGYHEALTHSTORE.COM | 301.378.8433



Powerful • Easy to Use
Effective • Excellent Results
Made in USA

PREMIUM
3-battery system. Most powerful vacuum erection system in the world. Includes: (4) Mach1 rings, CD, gel, hard plastic case

CHOICE
2-battery system. Quietest. Includes: (4) Mach 1 rings, CD, gel, hard plastic case

MANUAL
Includes: Effortless manual pump, (4) Mach 1 rings, CD, gel

* Large cylinder tubes available for all 3 systems

5 Year Warranty on all Pos-T-Vac Products



MACH 1 SUPPORT RINGS
- Sizes:
#5 (8 mm),
#6 (11 mm) [Most Common],
#7 (13 mm),
#8 (15 mm)
Price: \$21.00 ea



PREMIUM
Price: \$379.95



CHOICE
Price: \$249.95



MANUAL
Price: \$179.95



LOOP RING
Venous Leakage Band
Price: \$24.95



MAINTAIN
STAY FIRM LONGER
New
Adjustable Penile Constriction Loop
Price: \$24.95



Pjur® med
Lubricants for Men
Price:
\$19.95-\$24.95

STEP-BY-STEP VED INSTRUCTIONS

Create strong erections in less than 3 minutes!

Simply follow these easy step-by-step instructions. Call us with questions **301.378.8433**.



1 Attach the pump head (motor system).



2 Attach the pump head (manual system).



3 Connect the ring loader.




4 Slide the Constriction Ring.




5 Slide the Constriction Ring.



6 Slide the Constriction Ring.

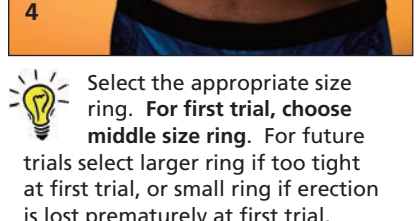


7 Apply Lubricant to obtain seal.

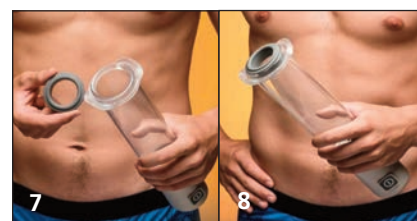

8 Apply Lubricant to obtain seal.



9 Apply Lubricant to obtain seal.

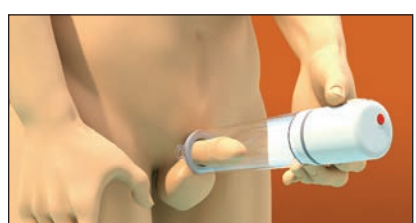


10 Apply device to hair-trimmed base of penis.




11 Apply device to hair-trimmed base of penis.



12 Firmly hold the device to create seal. Push the button (motor) to create vacuum.

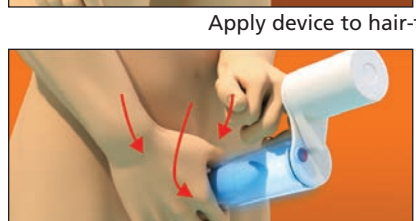

13 OR pull handle (manual) to create vacuum.

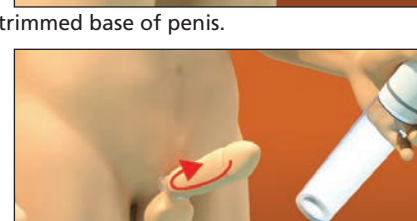

14 Release the ring when desired erection is achieved. Do not keep the ring on for more than 30 minutes.



15 Pull apart the ears of the constriction ring to detach.

Select the appropriate size ring. For first trial, choose middle size ring. For future trials select larger ring if too tight at first trial, or small ring if erection is lost prematurely at first trial.