



## Care and Handling for Nitrospray Spray Tips

- Before each use inspect the spray tips for cracks in the plastic hub, for bent shaft or shaft blockage.
- It is not necessary to sterilize the spray tips before using, as the operating temperature of the liquid nitrogen (-196° C) effectively destroys any and all known pathogens. However, tips should be kept and stored in a clean environment. If desired, cold soak sterilization is acceptable, and follow the manufacturer's directions. Do not autoclave the spray tips.
- Use of the spray tip protector will help to prevent damage to the spray tips and inadvertent contact with the patient and staff.