



MAINTENANCE INSTRUCTIONS

Post this in your sterilization room

How to lubricate high speed handpieces

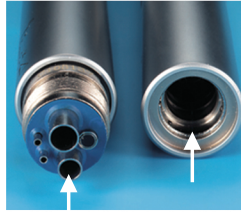

LUBE
HERE

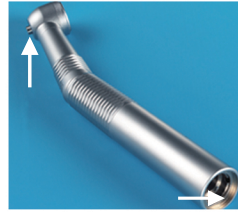

CAUTION

Do not place handpieces in
disinfectant baths.
Never submerge in water.
Do not place handpieces in
ultrasonic cleaners.

1 SPRAY

Using the correct lubrication tool, spray MK-dent Premium Spray lubricant into the drive line or hole after every use. Spray for approximately 2 seconds. The spindle of the handpiece needs to be lubricated once a week.



2 EXPEL

Remove the handpiece from nozzle and insert a bur. Loosely cover head with a paper towel. Run the handpiece to expel out debris and excess lubricant for 5-10 seconds. This should be done at the control unit in the operator or at the purging station in the sterilization room.



3 WIPE

Wipe down the exterior of the handpiece with a 2x2 lightly dampened fabric with pure rubbing alcohol only. Many over the counter "alcohol wipes" have other solutions added to them. This can cause spotting to the exterior and severe damage to the internal parts of the handpiece.



4 STERILIZE

Insert into a pouch, bag or wrap. Follow sterilizer manufacturer's instruction for autoclave/chemiclave. (Most products are sterilizable at temperature of 135°C/275°F)

Never remove components from sterilizer until dry cycle has ended.



5 ALLOW TO COOL

Remove the handpieces from the sterilizer and let them cool to room temperature. Running "hot" handpieces will severely reduce the life of the turbine.

Never cool the handpieces under running water or by covering / wrapping with wet cloth.



LUBRICANT ADAPTERS *Nozzles are reuseable and made out of aluminum for heavy duty use*



LT 1012
For KaVo
Designed to fit
KaVo brand
handpieces



LT 1013
E-type Low Speed
Designed to fit all
E-type air & electric
driven straight &
contra angle
handpieces



LT 1014
Cleaning Chucks
Designed to fit all
high speed chucking
mechanism to
prevent bur slippage



LT 1015
4 & 5 Hole
Handpieces
Designed to fit all
4 and 5 hole
handpieces



LT 1017
Sirona
Designed to fit
Sirona brand
handpieces



LT 1018
W & H
Designed to fit
W & H brand
handpieces



LT 1019
NSK QDI
Designed to fit
QDI brand
handpieces



LT 1020
NSK
Designed to fit
NSK Machilite/
Phatelus brand
handpieces



LT 1021
Star
Designed to fit all
Star low speed
and high speed
handpieces

ONLY
\$8.90
EACH



**HANDPIECE
SOLUTIONS**

6785 S. Eastern Ave., #11
Las Vegas, NV 89119
Toll Free: 1 (888) 488-3885
www.HandpieceSolutions.com

rev: 09/21